






Flexima® Uro Silk / Softima® Uro Silk

Flexima® Uro Silk / Softima® Uro Silk

Flexima® Uro Silk / Softima® Uro Silk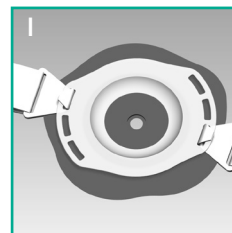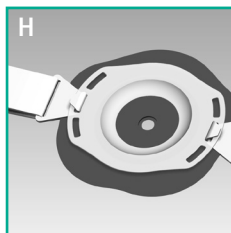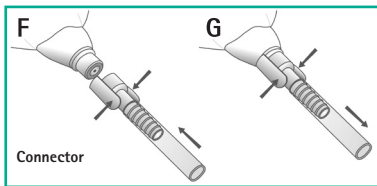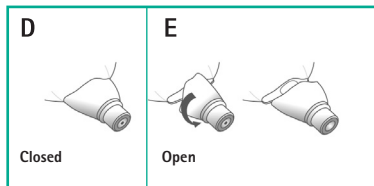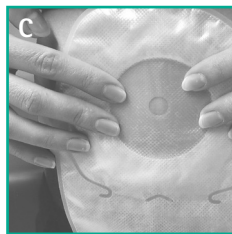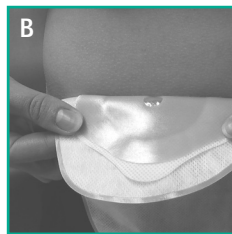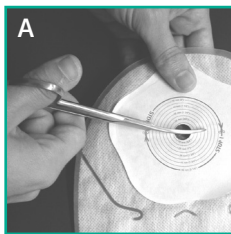

EN Instructions for Use

What Flexima® Uro Silk/ Softima® Uro Silk is and what it is used for:

Flexima® Uro Silk/ Softima® Uro Silk is a 1-piece drainable pouch for collection of urine from patients with a urostomy. Flexima® Uro Silk/ Softima® Uro Silk is available with a flat or a convex skin protector.

Flexima® / Softima® Uro Silk may be used concomitantly with B.Braun stoma care accessories.

Intended purpose and indication:

The product is intended for collection and containment of urine by patients with a urostomy. The product is not intended to be used on a skin or mucus membrane presenting a pathological change or change following disease or a wound.

Clinical benefits & performances: Ostomy appliances compensate for the loss of urinary continence.

Patient population: Urostomates adults or children above four years.

Contraindication: There are no formal contraindications for the use of Flexima® Uro Silk / Softima® Uro Silk.

Residual risks or side effects: Peristomal skin complications can occur while using Flexima® Uro Silk / Softima® Uro Silk.

1 Preparation of the skin

Wash your hands before starting the procedure.

Gently clean the peristomal area with lukewarm water. Avoid ether, alcohol or any other topical product which could interfere with base plate adhesion. Carefully dab dry the skin with soft and non-shedding cloths.

2 Applying the pouch

If you are using a Cut-To-Fit product, cut the central opening of the hydrocolloid adhesive to the size of your stoma (A) possibly with the help of the Stoma Guide. Do not cut beyond the maximum possible diameter and make sure that there is always a 2 to 3 mm margin around your stoma in order not to strangle or hurt the stomal mucosa. The pouch is supplied with the tap being closed and ready for use (D). Note: Pre-Cut products are not intended to be cut out.

Separate the pouch walls by crumpling it slightly to introduce air into it.

Grasp the pouch by the sides.

Peel off the release from the skin protector and position the

bottom of the flange opening around the base of the stoma (B). Press the bottom part of the skin protector smoothly onto the skin, then place and press onto the upper part taking care not to make folds.

In order to improve the adhesion, place hands over the skin protector to warm it up (C).

3 Emptying the pouch

Position yourself appropriately.

Point the tap towards the toilet bowl, slowly turn the tap counter-clockwise until the desired flow rate is obtained (E). After emptying the pouch, turn the tap clockwise back into its initial position (D).

4 Using uro connectors

The connection of the pouch to a B. Braun distal bag can require the use of the supplied uro connector. To clip on the uro connector, press the two wings and insert it into the pouch tap (F). Open the tap turning it half a turn counter-clockwise to allow urine to flow in the B. Braun distal bag. To un-clip, close the tap of the pouch by turning it half clockwise and pull out the uro connector by pressing the two wings (G).

5 Using B. Braun belt

If you wish to use a B. Braun belt, choose two loops that are diametrically opposed (H,I).

6 Removing the product

Pull slightly on the top of the skin protector and remove the pouch from the top to the bottom, while maintaining gentle pressure on the skin around the stoma with your other hand.

*Availability may vary from one country to another.

If you note any change on your stoma and/or peristomal skin, please contact your health care professional

National regulations for the destruction of potentially infectious materials must be observed.

- EN** Medical device as per 2017-745/EU
- DE** Medizinprodukt gemäß 2017-745/EU
- ES** Producto sanitario según el Reglamento (UE) 2017-745
- FR** Dispositif médical selon 2017-745/EU
- IT** Dispositivo medico secondo 2017-745/UE
- NL** Medisch hulpmiddel conform 2017-745/EU
- BG** Медицинско изделие съгласно 2017-745/EU
- CS** Zdravotnický prostředek dle nařízení (EU) 2017/745
- DA** Medicinsk udstyr i henhold til 2017-745/EU
- FI** Lääkinnällinen laite direktiivin 2017-745/EU mukaisesti
- ET** Meditsiiniseade 2017-745/EU kohaselt
- HU** Orvosi eszköz, a 2017/745/EU rendeletnek megfelelően
- JA** 2017-745/EUに従った医療機器
- LV** Medicīniskā ierīce saskaņā ar 2017-745/ES
- PL** Wyrób medyczny zgodnie z rozporządzeniem (UE) 2017/745
- PT** Dispositivo médico de acordo com a diretiva 2017-745/EU
- RO** Dispozitiv medical conform 2017-745/EU
- RU** Медицинское изделие согласно 2017-745/EU
- SK** Zdravotnícka pomôcka podľa normy 2017-745/EU
- SV** Medicinteknisk produkt enligt 2017-745/EU

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DK B. Braun Medical A/S
FI B. Braun Medical Oy
B.BRAUN AESCULAP JAPAN CO.,LTD
SE B. Braun Medical AB
B. BRAUN MEDICAL SAS | 26 Rue Armengaud | 92210 Saint-Cloud | FRANCE